EN\_User Manual PLS Spreader Bar

NL\_Handleiding PLS Juk

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---

All rights reserved.

No part of this publication may be reproduced, stored in a computer data file and/or made public in any form or in any manner, whether this is electronic, mechanical, through photocopying, recording or in any other way without the prior written permission of the publisher.

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# 1. PLS Spreader Bar

---

The PLS Spreader Bar can be attached to the PLS Central Suspension Point (CSP).

## 1.1. Safe Working Load table

---

Type + number	Safe Working Load	Suitable to;
2-point Spreader Bar (8102)	275 kg	AR-200 and AR-260/AR-275
4-point Spreader Bar (8103)	500 kg	AR-350 and AR-500

Slings may have another Safe Working Load than the Spreader Bar! Please, check this before lifting!

The PLS Spreader Bar may only be modified, to be added to the hole version of the CSP, by your distributor!

PLS and your distributor cannot be held responsible for any damage caused by accidental, inappropriate, improper, incorrect or injudicious use (described above and in section 1.4 of the User Manual Ceiling Hoist) of the PLS Spreader Bar or by insufficiently qualified persons. This also applies the use of a Spreader Bar or Sling made by any other company than PLS. In case of doubt, please contact your distributor.

## 1.2. Important

---

Plan the movement of the care recipient. Avoid situations in which the care recipient is in the sling without supervision. Before lifting, check that the care recipient is completely free of his / her environment so that body parts do not get trapped. Be careful with hoses and / or wires that are attached to the user. The user is not allowed to hold the carrying strap during movement because there is a chance that the loops will shift during lifting. This could cause constriction. Before lifting, make sure that the hand control and the cable are free of the Spreader Bar, the care recipient and any other object.

## 1.3. Explanatory label overview

---

2-Point Spreader Bar example:


- 2020-12 = Year – Month. Also, the batch number.
- REF: Reference number.
- 8102 = type number of the Spreader Bar, in this case a 2-Point Spreader Bar.
- SWL = Safe Working Load in kilogram.
- Manufacturer's name and address data.

## 1.4. Connect and release


---

The blue dot has a marking function. Only when the slot is in the same line as the dot, the hook can be connected and released on/off the Spreader Bar.

1. Hold the hook 90° to the Spreader Bar.
2. Place the opening of the hook over the pin of the Spreader Bar.
3. Pull back the hook and in the same movement, turn it back.
4. Please, reverse these steps to release the Spreader Bar.

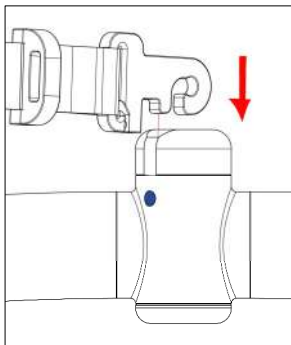*Figure 1, step 1 and 2*

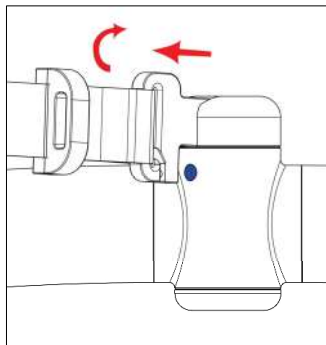*Figure 2, step 3*

## 1.5. Attaching the Sling

---

Be careful when attaching the lifting slings straps in/on the hooks! Use all the hooks to attach the sling!

Please make sure that the straps of the sling are secured in **all available** (2 or 4 hooks) hooks of the Spreader Bar!

Stay alert when lifting is started. The loops must stay in place!

*Figure 3, attaching the loops of the Sling on both hooks*

## 1.6. EC Declaration of conformity

---

1) We,

Patient Lifting Solutions (PLS) BV  
Barneveldsestraat 26a  
3927 CC Renswoude  
The Netherlands

2) Accept full responsibility that the devices in the product groups:

- **PLS Accessories**

*Contact PLS for the list of devices.*

3) Conform with the following standards:

- Regulation (EU) 2017/745

4) Supplementary information:

*Issued at:*  
**Renswoude, the Netherlands**

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