

# Basic Instruction Card AR-200



The Basic Instruction Card includes common instructions for optimal use of the Ceiling Hoist. This is not a replacement for the user manual! Before use, please read the manual carefully!

## Charging


The ceiling hoist charges when the hand control is placed in the wall charger.

- 1. Put the hand control in the wall charger.
- 2. Make sure that the charge indicator on the hand control lights up green.
- 3. If the hoist is been activated, check the control lightning of the hoist flashes.




## Control lighting and sound signal.


The control lights show the status of the batteries and or they need to be charged.

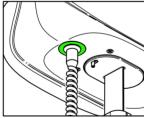

**Green**: batteries are fully loaded.

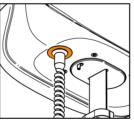


Orange: batteries need to be charged. Every 10 seconds a beep.

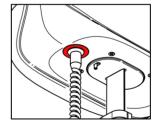

Red: the hoist need to be charged immediately! Every 1 second a beep. spreader bar (lifting belt) can be lowered.



#### Hand control

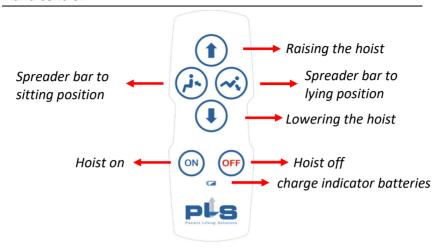




Before the lowering function starts to work, a weight, such as the spreader bar, should be hanged on the lifting belt.



Please, do not use the hand control and the spreader bar cable as drag cord.

## **Emergency break and emergency lowering function**

- 1. Pull the red emergency cord to immediately turn off the hoist.
- Keep pulling the cord, to slowly lower the spreader bar. The lowering function stops when the cord is being released. The hoist is now turned off.
- 3. To activate the hoist; press the button '*Hoist on*' on the hand control.

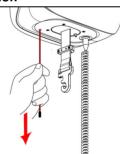

### Service

Please contact your distributor if you are dealing with questions, uncertainties and malfunction(s).

