



The Basic Instruction Card includes common instructions for optimal use of the Ceiling Hoist. This is not a replacement for the user manual! Before use, please read the manual carefully!

Charging


The ceiling hoist charges when the hand control is placed in the wall charger.

1. Put the hand control in the wall charger.
2. Make sure that the charge indicator on the hand control lights up **green**.
3. If the hoist is been activated, check the control lightning of the hoist flashes.




Control lighting and sound signal.


The control lights show the status of the batteries and or they need to be charged.

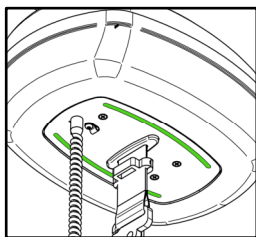

Green: batteries are fully loaded.

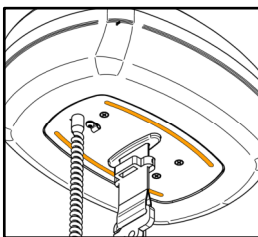


Orange: batteries need to be charged. Every 10 seconds a beep.

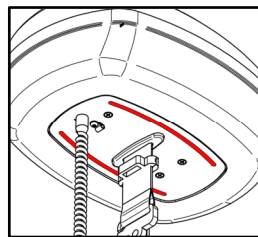

Red: the hoist need to be charged immediately! Every 1 second a beep. spreader bar (lifting belt) can be lowered.

Hand control

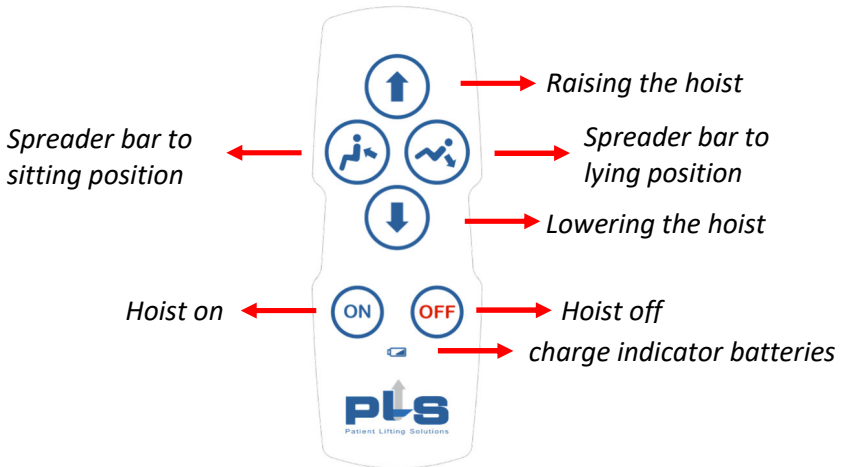


Before the lowering function starts to work, a weight, such as the spreader bar, should be hanged on the lifting belt.



Please, do not use the hand control and the spreader bar cable as drag cord.

Emergency break and emergency lowering function

1. Pull the red emergency cord to immediately turn off the hoist.
2. Keep pulling the cord, to slowly lower the spreader bar. The lowering function stops when the cord is being released. The hoist is now turned off.
3. To activate the hoist; press the button '**Hoist on**' on the hand control.

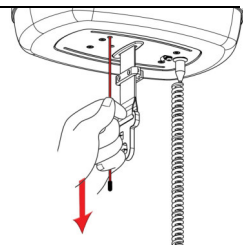

Service

Please contact your distributor if you are dealing with questions, uncertainties and malfunction(s).