Change Table Help Guide

CTE2 Nivano (Max Weight Capacity: 200KG)


1 Product description:

This change table is intended for showering and changing children and adults weighing up to 200kg, the table must not be used for any other purpose.

Pre-use checks and cautions:

- Check for obstructions or items that may become trapped when the change table is operated.
- Check the change table will raise and lower properly using the handset
- Never leave a person on the change table unattended.


3 Lowering the table

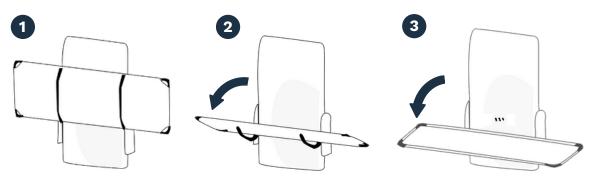

- To fold the table to the useable position, simply pull it down towards you.
- To stow the table lift and fold table until parallel with the cover.

Instruction for use

- Operation of the change table is via the hand control.
- To deploy the guard, lift and rotate the guard up into position and it will drop into the locked position.
- To stow the guard, lift the guard upwards, pull and rotate towards you and then lower it under the table to lock.

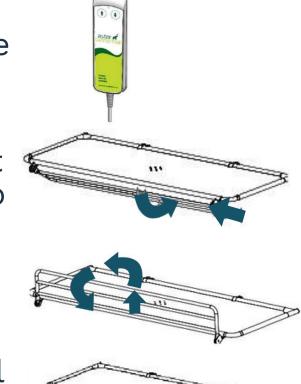

5 Cleaning

The change table may be cleaned using most mild cleaning agents or with a warm soapy water solution. The change table should be rinsed off after cleaning and dried off with a soft cloth.

Facility Management:

Name:

Number:



www.chshealthcare.com.au 1300 789 420