

# Change Table Help Guide

**CTX Nivano (Max Weight Capacity: 200KG)**

## **1 Product description:**

This change table is intended for showering and changing children and adults weighing up to 200kg, the table must not be used for any other purpose.

## **2 Pre-use checks and cautions:**

- Check for obstructions or items that may become trapped when the change table is operated.
- Check the change table will raise and lower properly using the handset
- Never leave a person on the change table unattended.


## **3 Lowering the table**



- To fold the table to the useable position, simply pull it down towards you.
- To stow the table lift and fold table until parallel with the cover.

## **4 Instruction for use**

- Operation of the change table is via the hand control
- To use the guard, pull on it from the stowed position and rotate it around to the upright position. Once upright it will drop and lock in place
- To stow, lift up until it can be rotated backward. Rotate under the table until the magnets hold it.

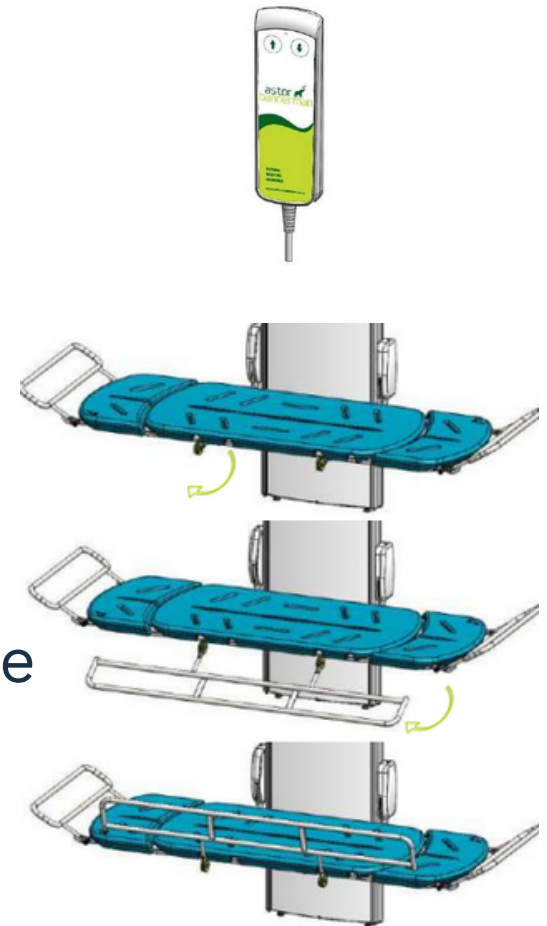

## **5 Cleaning**

The change table may be cleaned using most mild cleaning agents or with a warm soapy water solution. The change table should be rinsed off after cleaning and dried off with a soft cloth.

### **Facility Management:**

Name:

Number:



[www.chshealthcare.com.au](http://www.chshealthcare.com.au)

**1300 789 420**