

# Change Table Help Guide

**Astor Invincible (Max Weight Capacity: 200KG)**

## 1 Product description:

This change table is intended for showering and changing children and adults weighing up to 200kg, the table must not be used for any other purpose.

## 2 Pre-use checks and cautions:

- Check for obstructions or items that may become trapped when the change table is operated.
- Check the change table will raise and lower properly using the handset
- Never leave a person on the change table unattended.


## 3 Lowering the table



- To fold the table to the useable position, simply pull it down towards you.
- To stow the table lift and fold table until parallel with the cover.

## 4 Instruction for use

- Operation of the change table is via buttons on the main cover or front of the moving table
- To use the guard, pull on it from the stowed position and rotate it around to the upright position. Once upright it will drop and lock in place
- To stow, lift up until it can be rotated backward. Rotate under the table until the magnets hold it.

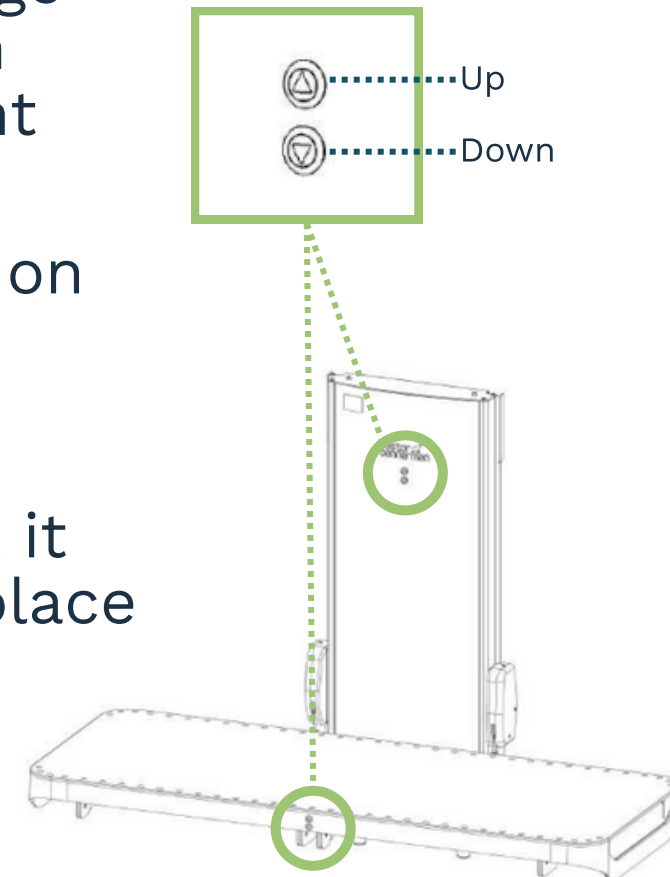

## 5 Cleaning

The change table may be cleaned using most mild cleaning agents or with a warm soapy water solution. The change table should be rinsed off after cleaning and dried off with a soft cloth.

### Facility Management:

Name:

Number:



[www.chshealthcare.com.au](http://www.chshealthcare.com.au)

1300 789 420